

**VIASYS**<sup>®</sup>  
HEALTHCARE

**LTV**<sup>®</sup> 1200

**EMERGENCY  
SETUP CARD**


**FOR 24 HOUR  
ASSISTANCE**

**800-754-1914**

STEP 1 CHOOSE A POWER SOURCE



**AC ADAPTER**  
(USE AC POWER WHEN AVAILABLE)

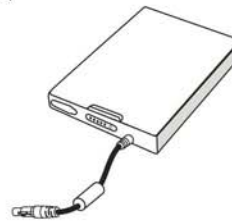


**LI ION BATTERY**



**LEAD ACID BATTERY**

STEP 2 CONNECT LTV TO POWER SOURCE

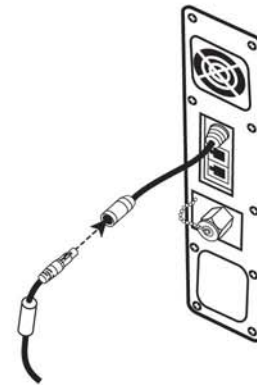

STEP 3 CHOOSE A CIRCUIT




OPEN BAG  
REMOVE PATIENT  
CIRCUIT AND  
**22mm ADAPTER**  
FROM BAG


HUMIDIFIER CIRCUIT / OPTIONAL



**PATIENT CIRCUIT ADULT**  
>20 kg  
>44 lbs

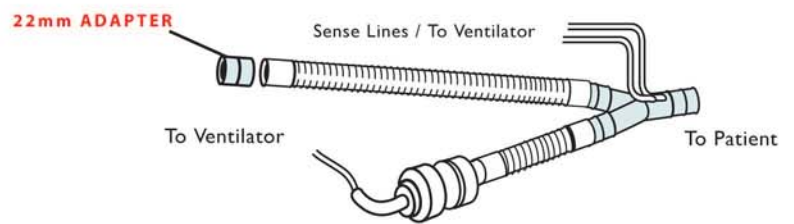


**PATIENT CIRCUIT PEDIATRIC**  
≤20 kg  
≤44 lbs

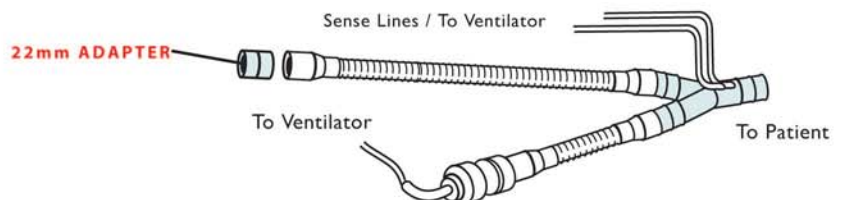


STEP 4 CONNECT CIRCUIT TO LTV

STEP A ATTACH 22mm ADAPTER (FOUND IN CIRCUIT BAG)


STEP B CONNECT CIRCUIT TO 22mm ADAPTER

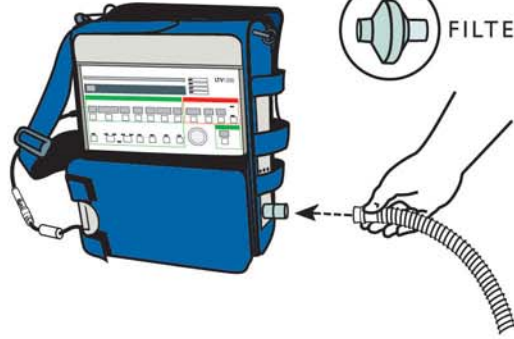

STEP 5 CONNECT SENSE LINES TO LTV

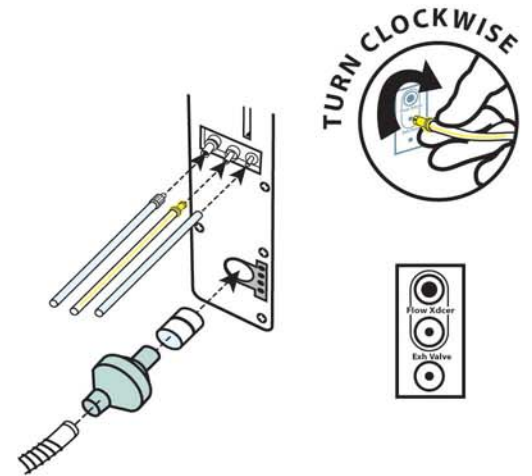


STEP 6 TURN LTV ON

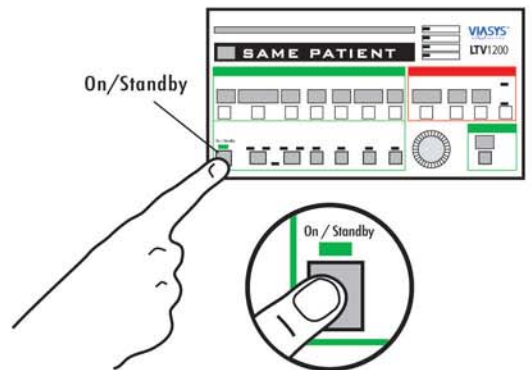

STEP 7 PATIENT SETTING

ONCE POWER IS ON SEE  
**SAME PATIENT** IN DISPLAY

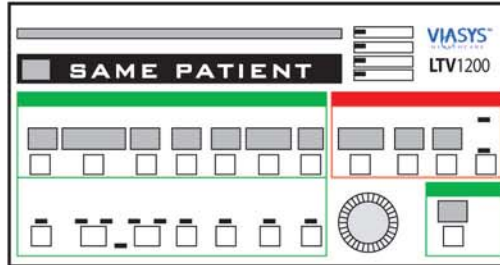


STEP 8 NEW PATIENT SETTING

STEP A ROTATE **SET VALUE KNOB**  
CLOCKWISE UNTIL  
**NEW PATIENT** IS DISPLAYED




STEP B PRESS SELECT WHEN  
**NEW PATIENT**  
IS DISPLAYED

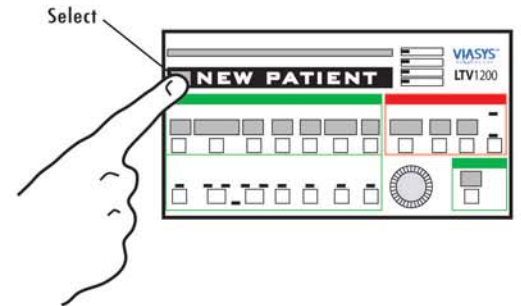


STEP 9 SET FOR PATIENT SIZE

STEP A ROTATE **SET VALUE KNOB** TO  
CORRECT PATIENT SIZE.  
(INFANT, PEDIATRIC OR ADULT)


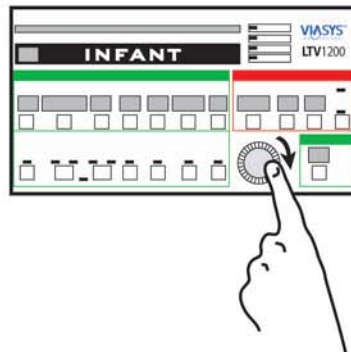


STEP B PRESS SELECT WHEN PROPER  
PATIENT SIZE IS DISPLAYED

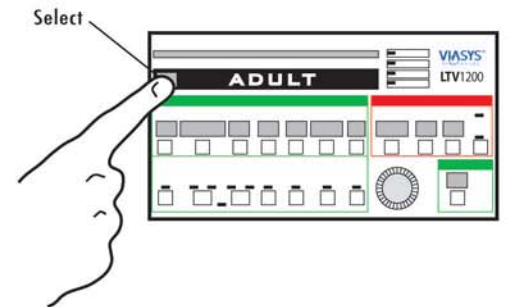

THE INITIAL SETTINGS MAY NEED TO BE ADJUSTED  
BASED ON THE PATIENT'S CLINICAL CONDITION.

STEP 10 CONNECT CIRCUIT TO ET-TUBE OR MASK

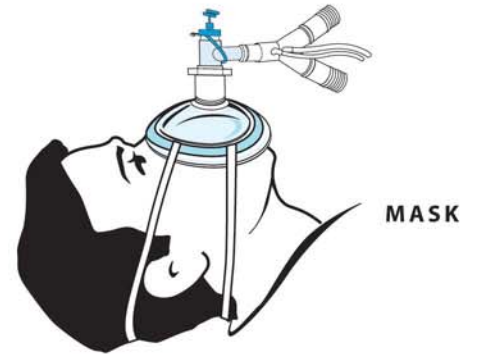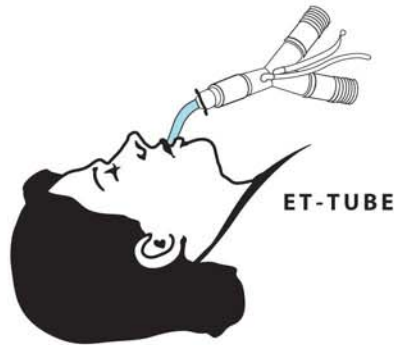

STEP 11 OPTIONAL // USING A MASK

CHOOSE A MASK (PEDIATRIC OR ADULT)



PEDIATRIC



ADULT


CONNECT ELBOW TO MASK



CONNECT TO PATIENT



STEP 12 OPTIONAL // USING AN HME AND/OR FLEX TUBE AS SHOWN FOR PATIENT COMFORT

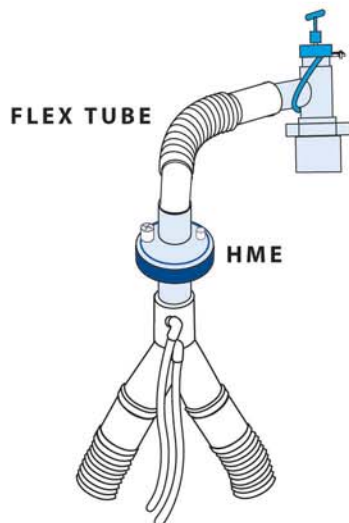

THIS EMERGENCY SETUP CARD IS INTENDED TO PROVIDE EMERGENCY INSTRUCTIONS FOR INITIAL SETUP OF THE LTV VENTILATOR. THE INITIAL SETTINGS MAY NEED TO BE ADJUSTED BASED ON THE PATIENT'S CLINICAL CONDITION. FOR COMPLETE INFORMATION REGARDING SETUP AND OPERATION OF THE VENTILATOR REFER TO THE LTV OPERATOR'S MANUAL

**LTV<sup>®</sup> I 200**

Clinical/Technical Support  
**1-800-754-1914** or  
763-398-8300  
[www.viasyshealthcare.com](http://www.viasyshealthcare.com)  
[www.pulmonetic.com](http://www.pulmonetic.com)

